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ENGLISH

INTENDED USE

ZAMED® Terra Trust Pregnancy Test is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin (hCG) in urine at a concentration level of 10 mIU/ml or greater to aid in the early detection of pregnancy. The test is designed for over-the-counter use.

PRINCIPLE OF TEST

ZAMED® Terra Trust Pregnancy Test is a rapid chromatographic immunoassay for the qualitative detection of human chorionic gonadotropin (hCG) in urine to aid in the early detection of pregnancy. The test utilizes a combination of antibodies including a monoclonal hCG antibody to selectively detect elevated levels of hCG. The assay is conducted by adding urine specimen to the specimen well of the test device and observing the formation of pink coloured lines. The specimen migrates via capillary action along the membrane to react with the coloured conjugate.

WARNINGS & PRECAUTION

- For in vitro diagnostic use and Self-testing use only.
- Check expiration date on package label before use. Do not use test kit beyond the expiry date.
- Inspect pouch for damage before use. Do not use if pouch is visibly damaged before opening.
- The test kit should not be reused.
- The test kit is moisture sensitive and should be used immediately after taking out of the pouch. When handling, avoid touching the test membrane.
- Urine specimens may be infectious; insure proper handling and dispose of all used reaction devices in normal household waste.

MATERIALS PROVIDED

- ZAMED® Terra Trust Pregnancy Test
- Instructions for use

MATERIALS NEEDED BUT NOT PROVIDED

- Clean glass or plastic container for specimens collection
- Timer

SAMPLE COLLECTION

A fresh urine specimen should be used, no special pre-treatment is necessary. Specimens should be collected in a clean glass or plastic container.

The specimen may be refrigerated (2-8 °C) and stored up to 2 days. For longer storage, freeze samples at -20 °C or below. Refrigerated samples should be allowed to come to room temperature and mixed thoroughly before assaying. Frozen samples should be thawed completely allowed to come to room temperature, and mixed thoroughly before assaying.

EXAMINATION PROCEDURE

Allow the test and the specimen to equilibrate to room temperature (15-30 °C) prior to testing

- To begin testing, open the sealed pouch by tearing along the notch. Remove the test kit from the pouch and use it as soon as possible.
- Hold the handle of the test with one hand. Use the other hand to remove the cap and expose the absorbent. Put the cap aside for now.
- Point the absorbent tip downward; place the absorbent tip in urine stream for at least 5 seconds to be thoroughly wet. Otherwise, you can collect your urine into a clean cup and dip half of the absorbent pad into the urine for at least 20 seconds.
- Close the unit again and lay it flat, on a straight surface. Wait until coloured stripes appear. This depends on the hCG concentration in the test sample. For all results, wait between 5 to 10 minutes to confirm the observation. Do not interpret the result after 30 minutes. It is important that the background is clear before reading the result.

Note: A low hCG concentration might result in a weak line appearing in the test region (T) after an extended period of time; therefore, do not interpret the result after 30 minutes.

INTERPRETATION OF RESULTS

POSITIVE:

Two pink lines appear, one in the results window. One line may be lighter than the other.

What does the result mean?

The test is positive and has detected the hCG pregnancy hormone in a concentration of 10 mIU/ml or more. You should talk to your doctor for prenatal care and advice.

NEGATIVE:

One pink line only appear in the control window.

What does the result mean?

The test is negative and no hCG has been detected in a concentration of 10 mIU/ml or more. However, if after seven days you still do not get your period, you should retest with another test. If you still get a negative result and your period has not started yet, you should see your doctor.

INVALID:

No line appears in C window.

What does the result mean?

The test is invalid. Use another test. You cannot assume you are pregnant or not. Take care to follow the instruction very well. Avoid the following:

- NOT keeping the absorbent stick pointing down
- NOT placing the results window facing away from your body
- NOT placing the absorbent point into your urine stream until it is thoroughly wet (for at least 3 seconds).
- You waited for more than 10 minutes.

STORAGE AND STABILITY

The test kit can be stable for at least 24 months at 4-30 °C and 21 days at 37 °C. The test must be used within 1 hour after opening the pouch, and stable for at least 100 days at transport conditions. The test kit should be kept away from direct sunlight, moisture and heat. The expiration dating was established under these storage conditions.

LIMITATIONS

- False negative results may occur when the levels of hCG are below the sensitivity level of the test. When pregnancy is still suspected, a first morning urine should be collected 48 hours later and tested.
- This test has been developed for testing urine samples only. The performance of this test using other specimens has not been substantiated.
- Very dilute urine specimens, as indicated by a low specific gravity, may not contain representative levels of hCG. If pregnancy is still suspected, a first morning urine specimen should be collected 48 hours later and tested.

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- Very low levels of hCG are present in urine specimen shortly after implantation. However, because a significant number of first trimester pregnancies terminate for natural reasons, a test result is weakly positive should be confirmed by retesting with a first morning urine collected 48 hours later.

- As with any assay employing mouse antibodies, the possibility exists for interference by human anti-mouse antibodies (HAMA) in the specimen. Specimens from patients who have received preparations of monoclonal antibodies for diagnosis or therapy may contain HAMA. Such specimens may cause false positive or false negative results.
- This test provides a presumptive diagnosis for pregnancy. A confirmed pregnancy diagnosis should only be made by a physician after all clinical and laboratory findings have to be evaluated.
- Because of hook effect, the test result may be indicate false negative when urine sample of 2-3 months pregnant directly detected.

MANUFACTURER / PRODUCER

Qingdao AiBo Diagnostic Co., LTD. NO. 2 Wannufangsi Road, Jinkou, Jimo, 266200 Qingdao People's Republic of China

EC AUTHORIZED REPRESENTATIVE

MedPath GmbH, Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

IMPORT AND DISTRIBUTION

Zamann Pharma Support GmbH, Siederstraße 7, 68623 Lampertheim, Germany

Mail: feedback@zamann-pharma.com
www.zamedu.com
www.zamann-pharma.com

ZAMED® is a brand of Zamann Pharma Support GmbH and in-house registered product brand for medical goods. It stands for products that have been fully tested and certified in the EU, by qualified and accredited bodies, and are under our direct control.

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